

Superior absorbency for tougher conditions[†]



Melgisorb Ag Ordering Information

Packaged sterile in single packs.

Product Code	Size	Pieces/box	Pieces/case	HCPCS
255050	2" x 2" (5 x 5 cm)	10	100	A6196
255100	4" x 4" (10 x 10 cm)	10	100	A6196
255150	6" x 6" (15 x 15 cm)	10	100	A6197
255200	8" x 12" (20 x 30 cm)	5	50	A6198
255600	1.2" x 18" (3 x 44 cm)	10	100	A6199

Indications

Melgisorb Ag is indicated for moderate to heavily exuding partial to full thickness wounds, such as:

- leg ulcers
- pressure ulcers
- diabetic/neuropathic ulcers
- graft and donor sites
- surgical wounds
- trauma injuries

This product can also be used under compression bandages. Because it contains alginate, Melgisorb Ag may assist in supporting the control of minor bleeding in superficial wounds. Melgisorb Ag is suitable for use, under medical supervision, on wounds that are critically colonized.

Contraindications

Melgisorb Ag is not indicated for use on the following:

- individuals with a known sensitivity to alginates or silver
- surgical implantation
- to control heavy bleeding
- for direct application on dry or lightly exuding wounds.

Precautions

- Systemic antimicrobial therapy or other infection treatment should be considered when wound infection is evident. Melgisorb Ag may be used, under medical supervision, in conjunction with systemic antibiotics.
- The dressing may adhere if used on dry or very lightly exuding wounds. If the dressing is not easily removed, moisten it with sterile saline solution prior to removal.
- The dressing performance may be impaired by use of petroleum-based ointments.

Ref 1, 2, 3. Data on file.



5550 Peachtree Pkwy, Ste 500, Norcross, GA 30092 | 800.882.4582 | www.molnlycke.com | www.safetac.com | www.mhcwoundcare.com

The Mölnlycke Health Care, Melgisorb, names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2008. Mölnlycke Health Care AB. 1.800.882.4582.

US011570806 - 270287 06.08

Superior
absorbency
for tougher
conditions†




● Melgisorb® Ag



 MÖLNLYCKE®
HEALTH CARE

† As compared to leading silver hydrofiber - data on file

Fact: Melgisorb[®] Ag absorbs up to 45% more than the leading silver hydrofiber dressing

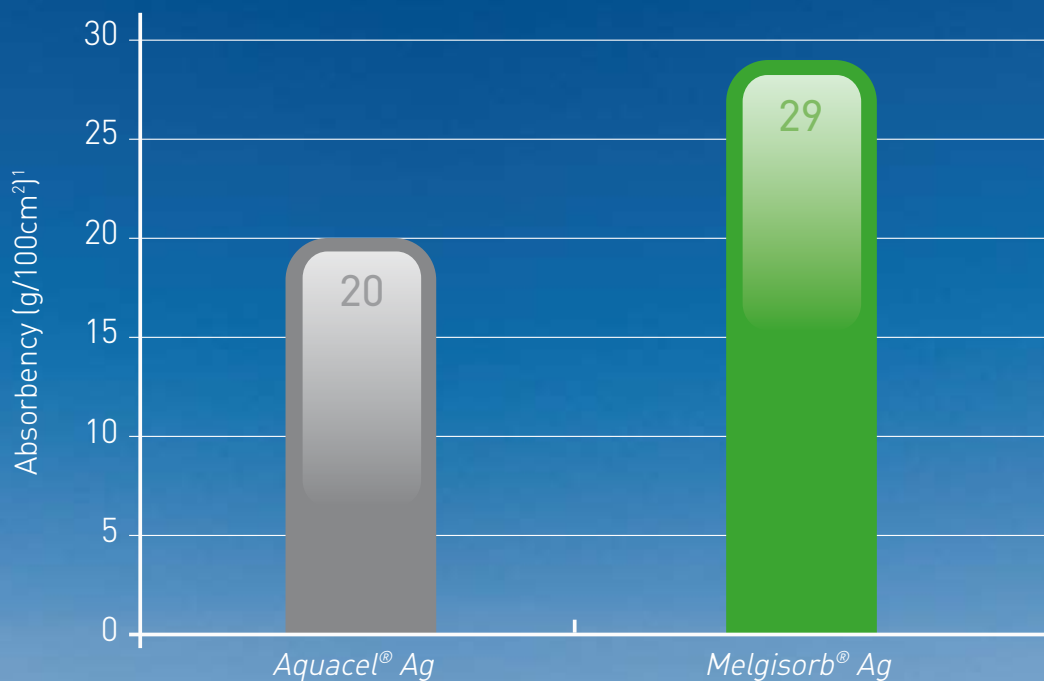


New Melgisorb Ag is a highly absorbent alginate dressing with CMC and ionic silver. Melgisorb Ag absorbs up to 45% more exudate than the leading silver hydrofiber dressing and kills bacteria for up to four days². The superb wet strength of the alginate fibers do not allow the dressing to disintegrate in the wound bed so it may be removed easily in one piece. For deep, highly exuding cavity wounds at risk of infection, choose the dressing designed to outperform: Melgisorb Ag.

Melgisorb Ag is ideal for deep, heavily exuding wounds prone to infection



Fact: High fiber strength allows removal in one piece

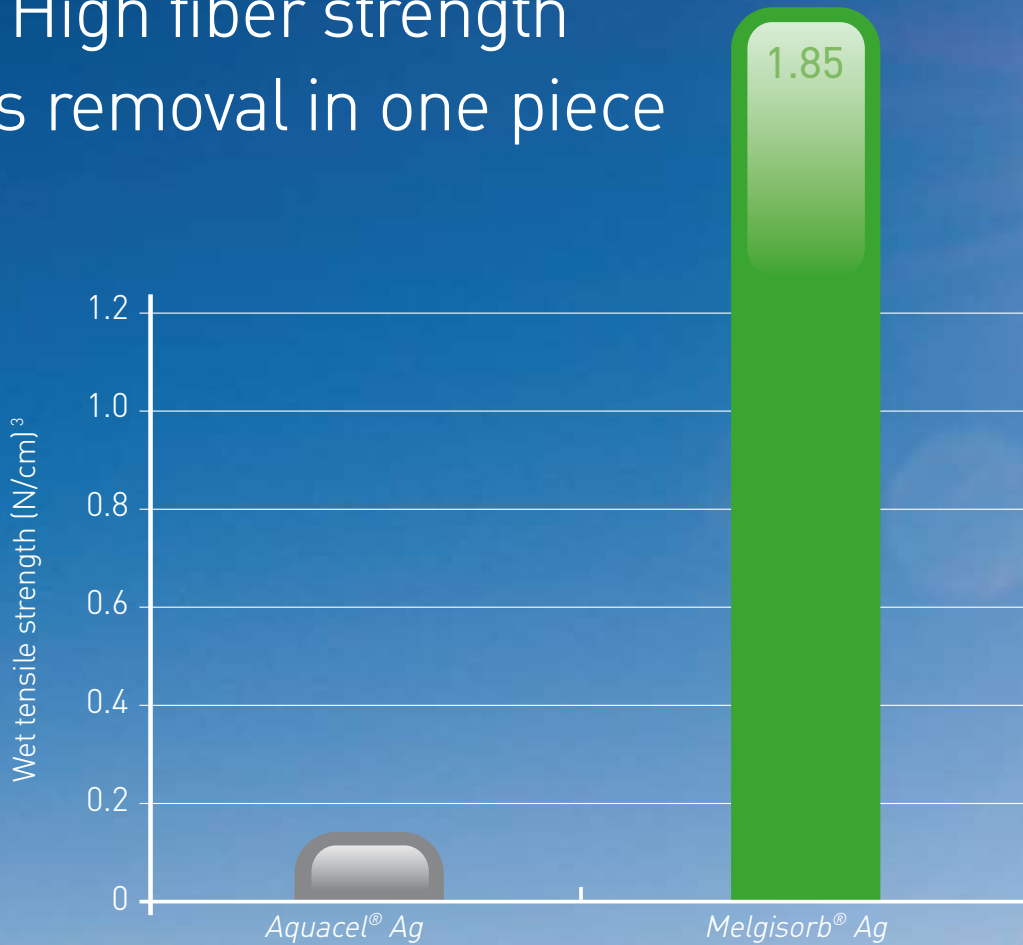

Melgisorb Ag releases silver ions in the presence of wound fluid. The silver ions protect the dressing from a broad spectrum of microorganisms over a period of up to four days. Melgisorb Ag is an effective barrier to bacterial penetration. Melgisorb Ag protects the wound and aids autolytic debridement therefore facilitating wound healing.

Handles more exudate – up to 45% more

Ensures easy dressing removal in one piece

Sustained antimicrobial activity