

Wound Dressing Selection Guide for Surgical Procedures

Procedure Application										
Description	Post-Op Dressings / Closed Surgical Wounds	Open Surgical Wounds (tertiary & dehiscence)	Around Pin Sites	Graft Sites	Donor Sites	Negative Pressure Therapy Applications	Amputations	Scar Management	Fixation	
Suggested Products	<ul style="list-style-type: none"> Mepore* Mepore*Pro Mepilex*BorderLite Mepilex*Border Post-Op Mepilex*Border Mepilex*Ag 	<ul style="list-style-type: none"> Mepitel* Mepilex* Mepilex*Border Mepilex*Ag 	<ul style="list-style-type: none"> Mepilex*Ag Mepilex* Mepilex*Lite 	<ul style="list-style-type: none"> Mepitel* Mepilex*Ag 	<ul style="list-style-type: none"> Mepilex* Mepilex*Ag 	<ul style="list-style-type: none"> Mepilex* Mepilex*Ag 	<p>AS CONTACT LAYER BETWEEN TISSUE AND DEVICE</p> <ul style="list-style-type: none"> Mepitel* <p>PERI-WOUND AREA</p> <ul style="list-style-type: none"> Mepilex*Lite Mepilex*Transfer 	<ul style="list-style-type: none"> Mepitel* Mepilex* Mepilex*Border Mepilex*Border Post-Op Mepilex*Transfer 	<ul style="list-style-type: none"> Mepiform* 	<ul style="list-style-type: none"> IVs Mepore*IV Mepore*Film FRAGILE SKIN Mepitac* FABRIC TAPE Mefix*
Treatment Objective	<ul style="list-style-type: none"> Manage exudate Reduce pain Cover and protect incision site from contamination Protect peri-wound skin 	<ul style="list-style-type: none"> Reduce pain Cover and protect incision site from contamination Manage exudate Protect peri-wound skin 	<ul style="list-style-type: none"> Reduce risk of infection Manage exudate 	<ul style="list-style-type: none"> Reduce pain and improve patient comfort Minimize shearing Decrease vulnerability to infection Reduce risk of contractures Manage exudate 	<ul style="list-style-type: none"> Reduce pain and improve patient comfort Increase mobility Decrease vulnerability to infection Facilitate re-epithelization Manage exudate 	<ul style="list-style-type: none"> Allow for sufficient negative pressure and drainage Protect tissue and structures Protect peri-wound skin Reduce pain and improve patient comfort 	<ul style="list-style-type: none"> Provide securement and stability to amputation site Manage exudate 	<ul style="list-style-type: none"> Improve patient comfort Minimize hypertrophy Improve appearance of scars Flatten scars Reduce redness of scars 	<ul style="list-style-type: none"> Securely fixate tubing, devices Reduce skin stripping or breakdown 	
Notations	<p>Eye Closure</p>	<p>Dressing Retention</p>	<p>Fill Space/ Absorb Exudate</p>	<ul style="list-style-type: none"> ◆ = Provides waterproof and bacterial barrier ◇ = Provides waterproof and bacterial barrier; and protection for fragile or sensitive skin ◆ = Provides antimicrobial action for infected wounds or wounds at risk 	<p>Indicates level of drainage/exudate</p> <ul style="list-style-type: none"> ○ = None ● = Scant ●● = Moderate ●●● = High 					

The information provided herein is not to be construed as the practice of medicine or substituted for the independent medical judgment of a patient's treating physician. This information, including but not limited to suggestions for product wear time, product selection and suggested use is based on generalizations and does not consider the unique characteristics of an individual's wound. Each patient's physician shall remain solely responsible for assessing the severity of patient wounds, determining the appropriate treatment, and managing treatment of the wound. For additional information, please refer to the applicable product insert or contact Molnlycke Health Care at 800.882.4582.

Dressings with Safetac TECHNOLOGY

Mepitel*
Soft silicone wound contact layer

Product Code	Size	Pieces/box	Pieces/case	HCPCS
290590	2" x 3" (5 x 7.5 cm)	10	60	A6206
290790	3" x 4" (7.5 x 10 cm)	10	50	A6206
291090	4" x 7" (10 x 18 cm)	10	50	A6207
292005	8" x 12" (20 x 30 cm)	5	30	A6208

Mepilex*Border Post-Op
Absorbent soft silicone self-adherent bordered foam dressing for surgical incisions

Product Code	Size	Pieces/box	Pieces/case	HCPCS
295800	4" x 8" (10 x 20 cm)	5	35	A6212
295900	4" x 12" (10 x 30 cm)	5	25	A6213

Mepilex*Border
Absorbent soft silicone self-adherent bordered foam dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
295200	3" x 3" (7.5 x 7.5 cm)	5	70	A6212
295300	4" x 4" (10 x 10 cm)	5	50	A6212
295400	6" x 6" (15 x 15 cm)	5	50	A6213
295600	6" x 8" (15 x 20 cm)	5	45	A6213

Mepilex*BorderLite
Absorbent soft silicone self-adherent bordered thin foam dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
281000	1.6" x 2" (4 x 5 cm)	10	70	—
281100	2" x 5" (5 x 12.5 cm)	5	65	A6212
281200	3" x 3" (7.5 x 7.5 cm)	5	70	A6212
281300	4" x 4" (10 x 10 cm)	5	50	A6212
281500	6" x 6" (15 x 15 cm)	5	50	A6213

Mepitac*
Soft silicone tape

Product Code	Size	Rolls/case	HCPCS
298300	3/4" x 118" (2 x 300 cm)	12	A4452
298400	1.5" x 59" (4 x 150 cm)	12	A4452

Mepilex*Ag
Absorbent soft silicone dressing with silver

Product Code	Size	Pieces/box	Pieces/case	HCPCS
287100	4" x 4" (10 x 10 cm)	5	70	A6209
287200	4" x 8" (10 x 20 cm)	5	45	A6210
287300	6" x 6" (15 x 15 cm)	5	25	A6210
287400	8" x 8" (20 x 20 cm)	5	20	A6211

Mepilex*
Absorbent soft silicone foam dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
294190	4" x 4" (10 x 10 cm)	5	60	A6209
294290	4" x 8" (10 x 20 cm)	5	30	A6210
294390	6" x 6" (15 x 15 cm)	5	30	A6210
294490	8" x 8" (20 x 20 cm)	5	30	A6211

Mepilex*Lite
Absorbent soft silicone thin foam dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
284090	2.4" x 3.4" (6 x 8.5 cm)	5	70	A6209
284190	4" x 4" (10 x 10 cm)	5	50	A6209
284390	6" x 6" (15 x 15 cm)	5	50	A6210
284590	8" x 20" (20 x 50 cm)	2	12	A6211

Mepilex*Transfer
Soft silicone exudate transfer dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
294890	6" x 8" (15 x 20 cm)	5	35	A6210
294592	8" x 20" (20 x 50 cm)	2	12	A6211

Mepiform*
Self-adherent silicone gel sheeting for scar management

Product Code	Size	Pieces/box	Pieces/case
293190	1.6" x 12" (4 x 30 cm)	5	25
293290	2" x 3" (5 x 7.5 cm)	5	30
293490	4" x 7" (10 x 18 cm)	5	25

Conventional Dressings

Mepore*
Self-adhesive absorbent dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
670800	2.5" x 3" (6 x 7 cm)	60	480	A6219
670900	3.6" x 4" (9 x 10 cm)	50	400	A6219
671000	3.6" x 6" (9 x 15 cm)	50	400	A6219
671100	3.6" x 8" (9 x 20 cm)	30	180	A6219
671200	3.6" x 10" (9 x 25 cm)	30	180	A6219
671300	3.6" x 12" (9 x 30 cm)	30	270	A6220
671400	3.6" x 14" (9 x 35 cm)	30	180	A6220

Mepore*Film
Self-adhesive transparent film dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
270600	2.4" x 2.6" (6 x 7 cm)	100	600	A6257
271500	4" x 5" (10 x 12.7 cm)	70	210	A6258
272500	4" x 10" (10 x 25 cm)	20	100	A6258
273000	6" x 8.5" (15 x 21.5 cm)	10	50	A6259

Mepore*IV
Transparent, breathable IV film

Product Code	Size	Pieces/box	Pieces/case	HCPCS
274000	2" x 2.2" (5 x 5.5 cm)	100	600	A6257
274200	3.2" x 3.6" (8 x 9 cm)	70	210	A6257
274400	4" x 4.3" (10 x 11 cm)	70	210	A6257

Melgisorb*
Absorbent calcium alginate dressing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
250600	2" x 2" (5" x 5" cm)	10	50	A6196
251100	4" x 4" (10" x 10" cm)	10	50	A6196
251500	4" x 8" (10" x 20" cm)	10	50	A6197
253000	12.5" (32 cm) Rope	5	50	A6199

Mefix*
Self-adhesive fabric tape

Product Code	Size	Rolls/case	HCPCS
310250	1" x 11 yds (2.5 cm x 10 m)	39	A4450
310500	2" x 11 yds (5 cm x 10 m)	20	A4450
311000	4" x 11 yds (10 cm x 10 m)	20	A4450
311500	6" x 11 yds (15 cm x 10 m)	20	A4450
312000	8" x 11 yds (20 cm x 10 m)	10	A4450

Mepore*Pro
Self-adhesive absorbent dressing with showerproof backing

Product Code	Size	Pieces/box	Pieces/case	HCPCS
670890	2.5" x 3" (6 x 7 cm)	60	480	A6203
670990	3.6" x 4" (9 x 10 cm)	40	320	A6203
671090	3.6" x 6" (9 x 15 cm)	40	320	A6203
671190	3.6" x 8" (9 x 20 cm)	30	180	A6203
671290	3.6" x 10" (9 x 25 cm)	30	180	A6203
671390	3.6" x 12" (9 x 30 cm)	30	270	A6203

Tubifast*
Lightweight, tubular dressing retention

Product Code	Line	Product	Width	Length
2434	RED	Small Limbs	1.4" (3.5 cm)	10 m
2436	GREEN	Small and medium limbs	2" (5 cm)	10 m
2438	BLUE	Large limbs	3" (7.5 cm)	10 m
2440	YELLOW	Extra-large limbs, heads, children's trunks	4.3" (10.75 cm)	10 m
2442	BEIGE	Adult trunks	7" (17.5 cm)	10 m
2444	PURPLE	Large adult trunks	10" (25 cm)	10 m

Tubifast* GARMENTS
Ready-to-wear for use in dressing retention & wet or dry wrapping

Product Code	Description
VESTS	
992007	Vest 6-24 Months
992008	Vest 2-5 Years
992009	Vest 5-8 Years
992010	Vest 8-11 Years
992011	Vest 11-14 Years
LEGGINGS	
992012	Tights 6-24 Months
992013	Leggings 2-5 Years
992014	Leggings 5-8 Years
992015	Leggings 8-11 Years
992016	Leggings 11-14 Years
SOCKS	
992017	Socks One Size
GLOVES	
5910	Gloves Child Small
5914	Gloves Child Medium
5915	Gloves Child Large/Adult Small
5916	Gloves Adult Medium
5917	Gloves Adult Large